




Plushape®

Plushape Pro®

-----Vacuum+RF+ Cavitation Slimming system

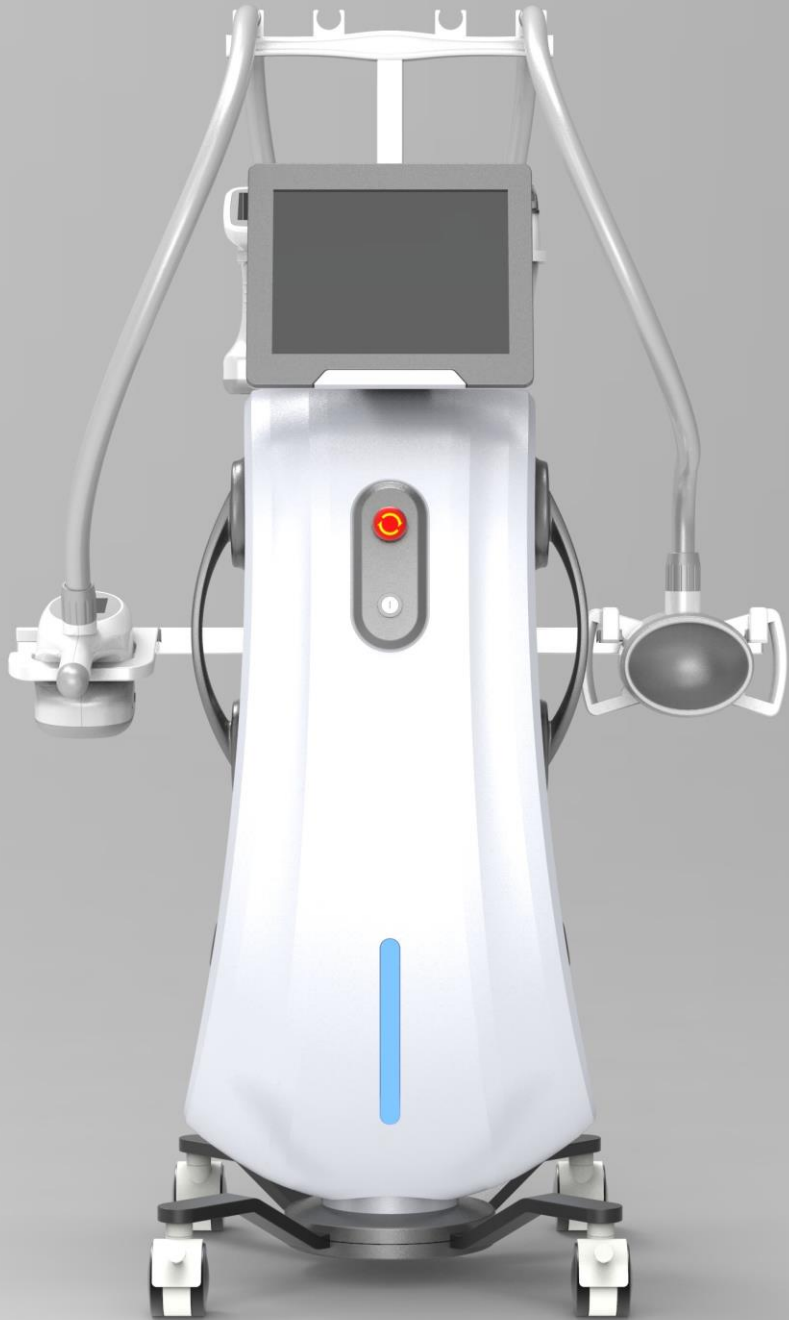

info@lasertell.com



+86-18610217037

Non-INVASIVE BODY CONTOURING

Skin tightening + Cellulite reduction

- ◆ Combine 5 technologies, eliminate cellulite
- ◆ Safe, efficient, comfortable, with no side effects
- ◆ With four different size handles for different treatment areas
- ◆ Only one treatment can see the result immediately
- ◆ The same parameter can be shown and adjust on handles directly



Working Principle

01



Pumping,relaxing,and pressing through vacuum vacuum suction

02



03



Reuse 940nm infrared to effectively reduce fat cells

04



Then use the roller or massage to dissolve deep fat,clear the lymph,eliminate toxins,eliminate edema

05



Use 10MHz RF to tighten and enhance skin after weight loss

06



Finally,the orange peel is faded, the curve is attractive, and the skin is smooth and smooth

Host details



- ◆ Profitable business driver-high patient demand for cellulite & bodycontouring treatments
- ◆ Customized designed vacuum pump,strong power low noise
- ◆ 10MHz RF for skin lifting,wrinkle removal,obvious result
- ◆ Rollers come with 4 direction with different effect
- ◆ Individual control in each handle,more convenient operation
- ◆ 10.4 inch color LCD touch screen,intelligent setting

Handle details



Handle 1

Size:300*125*105mm



Treatment area:
Back, Abdomen,
Buttocks, Thigh,
Shank



Handle 2

Size:230*75*100mm



Treatment area:
Arm, Neck



Handle 3

Size:180*45*45mm



Treatment area:
Face, Eye



Handle 4

Size:230*110*110mm



Treatment area:
Waist

Handle details

Handle 1 : vacuum with mechanical roller and radio frequency with infrared

Handle 2: vacuum with roller, radio frequency with infrared


Handle 3: vacuum with radio frequency and infrared

Handle 4: big cavitation handle with 635nm diode laser

Treatment Name	Handles	Total time of one treatment	Treatment times	Interval of two times treatment
Facial	❖ Handle 2 for 10 minutes ❖ Handle 3 for 15-20 minutes	30 minutes	6-10 times	5-7 days
Eyes	❖ Handle 3 for 20 minutes	20 minutes	6-10 times	5-7 days
Neck	❖ Handle 2 for 5 minutes ❖ Handle 3 for 10-15 minutes	20 minutes	6-10 times	5-7 days
Whole Body Slimming	❖ Handle 4 Cavitation handle 30 minutes ❖ Handle 1 for 20 minutes	50 minutes	6-8 times	3-4 days
Parts slimming	❖ Handle 4 Cavitation handle 20 minutes ❖ Handle 1 for 15 minutes	30-40 minutes	6-8 times	3-4 days

Applications

- ◆ Eyes wrinkle removal
- ◆ Facial lifting skin lifting
- ◆ Cellulite reduction
- ◆ Fat removal
- ◆ Body conturing & body tightening
- ◆ Build woman S curve mould perfect figure
- ◆ Skin relex for massage, reliable body pain
- ◆ Body detox, promote blood and lymphatic
- ◆ Circulation
- ◆ Breast lifting

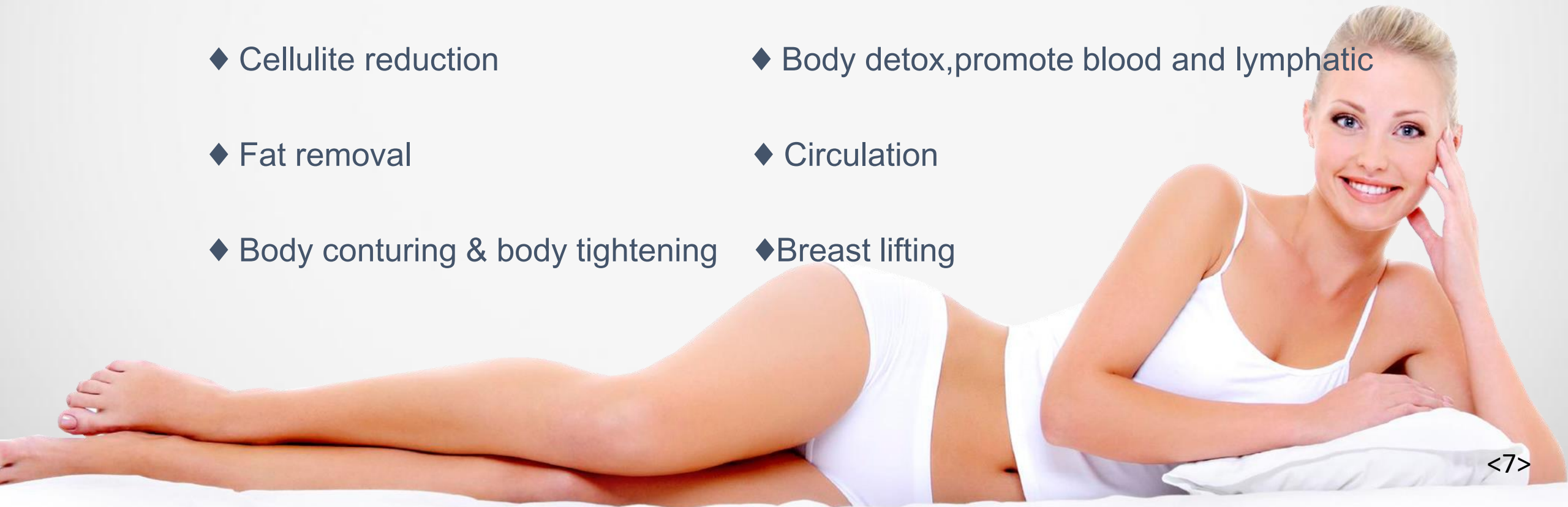

Before & After

Plushape®



Before & After

Plushape®



Before & After

Plushape®



Technical Specification	
RF Power	100W
Infrared Power	5-20W
Vacuum	0-100kPa
Size of Treatment Handle	Body:80mm*40mm Arm:12mm*40mm Face:14mm*3mm
Infrared Wavelength	650nm
Vacuum mode	Continuous
RF Frequency	10MHz
Vacuum Pressure	30-95Kpa
Roller Speed	0-36r/m
Voltage	AC220V±10% ; 50Hz/AC110V±10% ; 60Hz

Thank you!

 info@lasertell.com

 +8618610217037

 www.lasertell.com

 Room 308, No.12, Jingsheng South 2nd
Road, Tongzhou District, Beijing, China